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- (1) Lodde, M., Marberger, M. et al.: Detection of upper urinary tract transitional cell carcinoma with ImmunoCyt^{MC}: A Preliminary Report. *Urology*, 58: 362, 2001
- (2) Fradet, Y., Lockhart, C.: Performance characteristics of a new monoclonal antibody test for bladder cancer: ImmunoCyt^{MC}. *Canad J Urol*, 4: 400, 1997
- (3) Mian, C., Marberger, M. et al.: ImmunoCyt^{MC}: A new tool for detecting transitional cell cancer of the urinary tract: *J Urol*, 161: 1486, 1999
- (4) Carter, M., Kinahan, T. et al.: Sensitivity of ImmunoCyt in Patients with Transitional Cell Carcinoma or the Bladder. Poster Presentation CUA June 2000
- (5) Olsson, H., Zackrisson, B.: ImmunoCyt^{MC} a Useful Method in the follow-up Protocol for Patients with Urinary Bladder Carcinoma. *Scand J Urol Nephrol* 35: 280, 2001
- (6) Tétu, B., Tiguert, R. et al.: Le test ImmunoCyt: utilisation en routine pour le diagnostic et le suivi des tumeurs transitionnelles de la vessie. *Ass. des Pathologistes du Québec*, Oct. 2001
- (7) Rambeaud, J.J., Pfister, C. et al.: The Effectiveness of ImmunoCyt Test System in the Diagnostic of Bladder Cancer: French Multicentric Study. *EAU* 2002